
Intoxilyzer® 8000

Initial Training

Issue date: August 2015

Intoxilyzer® 8000


The Intoxilyzer® 8000 is a mobile breath alcohol instrument manufactured by CMI, Inc. in Owensboro, Kentucky, which uses infrared spectroscopy as its method of analysis. For comparison, the Preliminary Breath Testers (PBT's) used in North Dakota, the Intoxilyzer® S-D5 and Intoximeters Alco-Sensor FST®, use an electrochemical fuel cell for analysis.



INTOXILYZER® 8000 PARTS

Operators should be familiar with the key instrument components:

1. Breath Hose
2. Start Test Button
3. Internal Printer Compartment
4. Ethanol Gas Standard Connection
5. 2D Card Reader

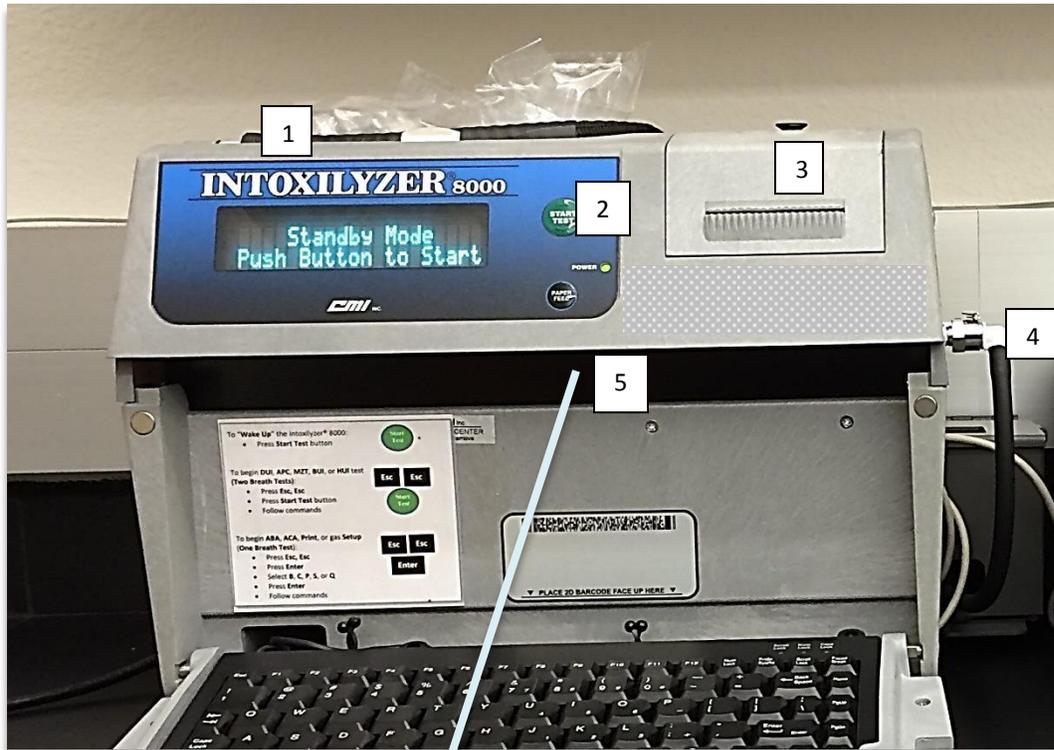

6. Printer Connector (USB)
7. Modem Connector
8. AC Power Switch
9. Battery Power Switch
10. Power Cord
11. Lock box with Ethanol Gas Standard inside



NOTE: Both the main power switch AND the battery power switch need to be turned on when in use. When the instrument will no longer be used for a period of time (i.e. building remodel) or it needs to be transported to the Crime Laboratory, BOTH the main power and battery power switches need to be turned off. This prevents internal circuit damage due to unintentional discharge of the external battery.

INTOXILYZER® 8000 INSTALLATION

The Intoxilyzer® 8000 must be installed by a certified Field Inspector prior to running any tests at an agency or for sobriety checkpoints. If the instrument is moved to a different location a Field Inspector needs to re-install it. Field Inspectors require more training than the Initial Intoxilyzer® 8000 class allows.

Once properly installed, the Intoxilyzer® 8000 will display Ready Mode. Below is a list of other operational modes one may see on the display.

OPERATIONAL MODES

1. Not Ready Mode: The Intoxilyzer® 8000 is not ready for a test. It enters Not Ready Mode when the AC power switch and the battery power switch are first turned on. The instrument will take approximately 20 minutes to warm up with the last 5 minutes of the warm up period displayed as a countdown.
2. Ready Mode: The Intoxilyzer® 8000 is ready to run a subject test. Ready Mode will be displayed after the initial warm up period and after the one minute countdown to get out of Standby Mode.
3. Standby Mode: If the Intoxilyzer® 8000 is not used for about 30 minutes it will suspend operation to save power and go into Standby Mode. An operator cannot run a test in this mode. To return to Ready Mode, press the green Start Test button. Ready Mode will display after a one minute countdown shown on the display.
4. Disabled Mode: The Intoxilyzer® 8000 has been disabled by the Crime Laboratory and cannot be used to conduct breath alcohol tests.

TESTS

It is the officer's decision which chemical test to give a subject for alcohol testing; whether the test is a breath, blood, or urine test. If a breath alcohol test is chosen, there are two different test sequences the Intoxilyzer® 8000 operator can use depending on the situation:

1. Custom Mode Sequence (CMS, two-breath test),
2. Single Breath Sequence (ABA)

Custom Mode Sequence (CMS)

This is the evidential two-breath test the majority of operators will be using. The Custom Mode Sequence (CMS) is a North Dakota customized sequence of steps which meet both the recommendations of the National Safety Council for evidentiary breath alcohol testing and the requirements for the Crime Laboratory's *Approved Method to Conduct Breath Tests with the Intoxilyzer® 8000*.

A CMS test must be run, and the approved method followed, when under arrest for Driving Under the Influence (DUI), Actual Physical Control (APC), Snowmobiling Under

the Influence (SUI), Off Highway Vehicle (OHV), Minor Zero Tolerance (MZT), Hunting Under the Influence (HUI), and Boating Under the Influence (BUI).

A valid breath alcohol test in Custom Mode Sequence must contain all of the following steps and in the following order:

1. Diagnostic
2. Room Air
3. Subject Test
4. Room Air
5. Standard Gas
6. Room Air
7. Subject Test
8. Room Air

A more in depth explanation of each step will be discussed in the following section.

Begin CMS Test:

The officer chose to give the subject a breath alcohol test instead of a blood or urine test and the Intoxilyzer® 8000 was installed by a Field Inspector sometime prior to testing. When arriving at the instrument's location the certified Intoxilyzer® 8000 operator will notice the display reads either:



The instrument is NOT ready for a test. Press the Start Test button for the one minute countdown to warm up the instrument.



Ready Mode will display after a diagnostic test is complete.

Or,



Indicates the instrument is ready for a test.



Remember: Press the Start Test button if the instrument is in Standby Mode. Operators call in to the lab stating that the instrument "froze up" when in fact the operator forgot to press the Start Test button first.

When in Ready Mode, press the Escape key twice (ignore “Enter Password:”) and then the Start Test button.

Escape, Escape, Start Test button



The display now reads:




The easiest way to scan the operator card is to place it across the top row of the keyboard and underneath the 2D card reader. If it doesn't scan remove it and place it down again.

The operator also has the option to press the enter key to manually enter data.

Scan operator ID card, verify information* displayed, and press Enter to accept data and move forward.

*Operator information to enter and/or verify includes: Operator's number, last name, and first name.

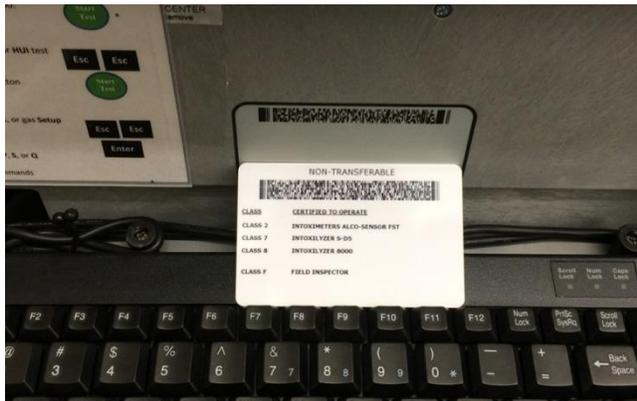

A common mistake is scanning the subject's driver's license instead of the operator's own card. The instrument will not mistakenly scan a driver's license, but instead will beep every time the license is placed under the card reader.



The operator should ascertain that the subject has had nothing to eat, drink, or smoke within 20 minutes prior to the collection of the breath sample. If 20 minutes has not been met, do not proceed until it has.

Press Y and Enter.



Currently, four copies are needed:

1. Operator file
2. Subject
3. Prosecutor
4. Certified copy with the Report and Notice to the Dept. of Transportation

Enter the number of copies and press Enter.



It is acceptable to print one copy, sign it, and make copies of the signed test record (instead of signing four copies). However, make sure your copier is printing legibly or there may be issues in court.



Scan the subject's driver's license the same way as the operator's card.

The operator also has the option to press the enter key to manually enter data.

Scan card, verify information* displayed, and press Enter to accept data and move forward.

*Subject information to enter and/or verify includes: Subject's last name, first name, middle name, date of birth, sex, and weight.



Test Reason choices include: DUI, HUI, MIC, OHV, OTH (other), SUI, WRK (work release), and BUI.

Scroll through with the arrow keys. Choose the test reason by pressing Enter.

The display reads as below. **Verify data, correct if needed, and press Enter.**



If a citation number is not available enter "NA" or leave blank.



Enter the two digit number for county of arrest.



If needed, the operator may enter "Y" to correct any data. Otherwise, press Enter to move forward.




If all criteria are met for the Diagnostic test, testing continues.



The instrument is purging the breath hose and sample chamber.



The instrument is determining a baseline.

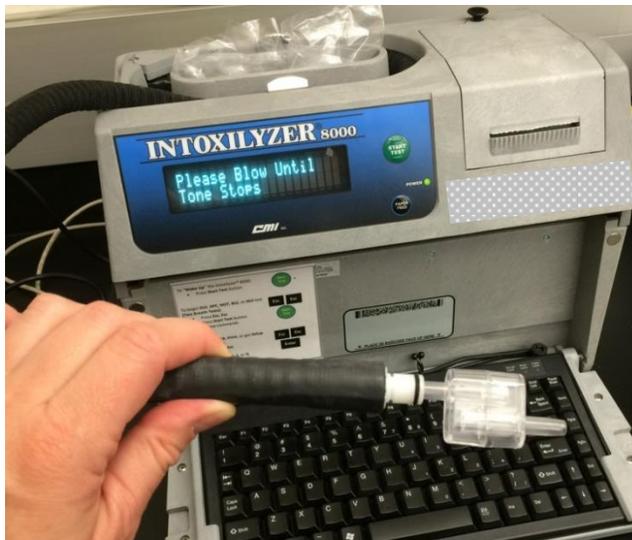

The display reads "Please Blow Until Tone Stops". The subject will have 3 minutes to provide an adequate breath sample.

Attach a clean mouthpiece and instruct the subject to blow into the mouthpiece until the instrument displays the zero to the left of the decimal point (the tone will also stop).



Discard the used mouthpiece. Do not keep it attached to the breath hose.



If a clean mouthpiece was used, **Press “Y” and then Enter.** There will be a predetermined timed delay of ≈ 50 seconds.



The instrument is purging the breath hose and sample chamber. There will be another predetermined timed delay of ≈ 50 seconds.



The instrument is determining a baseline.



The gas standard is being analyzed and will display the result. The current Ethanol Gas Standard is 0.080 alcohol concentration (AC). **The acceptable range is 0.075 – 0.085 AC.**



The instrument is purging the breath hose and sample chamber. There will be another predetermined timed delay of ≈ 50 seconds.



The instrument is determining a baseline.



The subject will have 3 minutes to provide an adequate breath sample.

Do not reuse a used mouthpiece.

Attach a clean mouthpiece and instruct the subject to blow into the mouthpiece until the instrument displays the zero to the left of the decimal point (the tone will also stop).





If a clean mouthpiece was used, **Press “Y” and then Enter.**



The instrument is purging the breath hose and sample chamber for a final time.

The difference between the first and second subject breath samples is calculated. The display should read either:



“Difference OK” means the difference between the two breath samples is equal to or less than 0.020 g/210 L (20 digits) of expired breath.

Or,



“Difference Too Great” means the difference between the two breath samples is greater than 0.020 g/210 L (20 digits) of expired breath.

The instrument will begin to print the test record and the display returns to Ready Mode.

Sign all copies of the test record (Form 106-I8000).

Johnny Utah
Operator Signature
JOHNNY UTAH

Remarks:

Form 106-I8000

If the instrument has not been used for a period of time (20-30 minutes), it will return to Standby Mode.

The following test records are examples of valid breath alcohol tests run in CMS.

Two adequate breath samples:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = BSPD 8164.13.00 06/09
07/10/2018 10:49

Test	AC	Time
01 Diagnostic	OK	10:50
02 Room Air	0.000	10:50
03 Subject Test 1	0.088	10:51
04 Room Air	0.000	10:53
05 Std. Gas	0.079	10:54
06 Room Air	0.000	10:55
07 Subject Test 2	0.087	10:56
08 Room Air	0.000	10:57
09 Reported AC	0.087	10:56

Difference OK
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 59
Expiration Date = 08/01/2018
County = 08 Oper No. = 777777

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Form 106-I8000

Line 09, Reported AC, will report the lower of the two AC values and the time that sample was provided.

VALID TEST



If the subject refuses to provide a breath sample for one of the two samples during a test, the one adequate breath sample shall be reported and is legally admissible.

One refusal and one adequate breath sample or vice versa:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = BSPD 8164.13.00 06/09
07/09/2018 11:12

Test	AC	Time
01 Diagnostic	OK	11:13
02 Room Air	0.000	11:13
03 *Subject Test	0.000*	11:16
04 Room Air	0.000	11:19
05 Std. Gas	0.078	11:20
06 Room Air	0.000	11:21
07 Subject Test 1	0.089	11:22
08 Room Air	0.000	11:23
09 Reported AC	0.089	11:22

No RFI Detected

*Deficient Sample - Value Printed was Highest Obtained

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 59
Expiration Date = 08/01/2018
County = 08 Oper No. = 777777

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Form 106-I8000

The subject refused to provide the first breath sample, but provided an adequate sample for the second breath, which will be the Reported AC on line 09.

**VALID
TEST**

One adequate sample and one refusal (with some ethanol present) or vice versa:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = BSPD 8164.13.00 06/09
07/09/2018 11:00

Test	AC	Time
01 Diagnostic	OK	11:00
02 Room Air	0.000	11:01
03 Subject Test 1	0.090	11:01
04 Room Air	0.000	11:03
05 Std. Gas	0.079	11:05
06 Room Air	0.000	11:06
07 *Subject Test	0.072*	11:09
08 Room Air	0.000	11:10
09 Reported AC	0.090	11:01

No RFI Detected

*Deficient Sample - Value Printed was
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 59
Expiration Date = 08/01/2018
County = 08 Oper No. = 777777

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.


Operator Signature
DEB KASHUR

Remarks:

Form 106-I8000

The subject provided an adequate sample for the first breath, but for the second breath the subject blew a little and showed ethanol is present, but quit blowing before an adequate sample could be accepted. Therefore, the reported AC will be the adequate sample of 0.090.

**VALID
TEST**

Single Breath Sequence (ABA)

ABA = Air Blank, Breath, Air Blank.

As the name implies, this is a single breath test run on a subject for testing other than DUI or other driving offenses. For example, this test may be used for MIC, Parole/Probation, Work Release, or a parent's request.

This test does NOT follow the Crime Laboratory's *Approved Method to Conduct Breath Tests with the Intoxilyzer® 8000* and, therefore, is not allowed as an evidentiary test for driving offenses.

Begin ABA test:

Again, when the subject is brought to the Intoxilyzer® 8000 for testing, the display will read either:



The instrument is NOT ready for a test. Press the Start Test button for the one minute countdown to warm up the instrument.



Ready Mode will display after a diagnostic test is complete.

Or,



Indicates the instrument is ready for a test.

The operator needs to get into Level 1 in the Intoxilyzer® 8000. Do this by pressing the Escape key twice (ignore “Enter Password:”) and then press Enter.

Escape, Escape, Enter



*Operators often mistakenly run tests in ABA mode when they wanted to run a two breath test in CMS. Remember, to enter CMS press Esc, Esc, **Start Test** button, not the Enter key.*

This takes you to Level 1 with the following options:



B = Breath Test (ABA)

C = Calibration Check

P = Print Test

S = Configure Standard


Q = Quit

Press B then Enter to begin the test.

The display now reads:



The easiest way to scan the operator card is to place it across the top row of the keyboard and underneath the 2D card reader. If it doesn't scan take it away and place it down again.

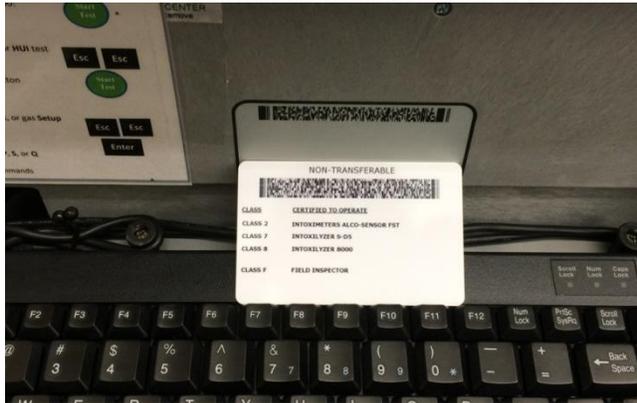

The operator also has the option to press the enter key to manually enter data.

Scan operator ID card, verify information* displayed, and press Enter to accept data and move forward.

*Operator information to enter and/or verify includes: Operator's number, last name, and first name.



Currently, three copies are needed:

1. Operator file
2. Subject
3. Prosecutor

Enter the number of copies and press Enter.



Scan the subject's driver's license the same way as the operator's card.

The operator also has the option to press the enter key to manually enter data.

Scan card, verify information* displayed, and press Enter to accept data and move forward.

*Subject information to enter and/or verify includes: Subject's last name, first name, middle name, date of birth, sex, and weight.



Test Reason choices include: DUI, HUI, MIC, OHV, OTH (other), SUI, WRK (work release), and BUI.

**Scroll through with the arrow keys.
Choose the test reason and press Enter.**



If a citation number is not available enter "NA" or leave blank.



Enter the two digit number for county of arrest.



If needed, the operator may enter "Y" to correct any data. Otherwise, press Enter to move forward.



The instrument is purging the breath hose and sample chamber.



The instrument is determining a baseline.



The subject will have 3 minutes to provide an adequate breath sample.

Attach a clean mouthpiece and instruct the subject to blow into the mouthpiece until the instrument displays the zero to the left of the decimal point (the tone will also stop).





The instrument is purging the breath hose and sample chamber for a final time.

The instrument will begin to print the test and again display Level 1.

Sign all copies of the test record (Form 106-I8000).

A scan of a form titled "Form 106-I8000". It features a signature "Stephen Caracappa" in cursive, followed by the printed text "Operator Signature" and "STEPHEN CARACAPPA". Below this is a section labeled "Remarks:" which is currently blank.

At this point the operator can press Q then Enter or let the instrument automatically go back to Ready Mode.



ABA Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = BSPD 8164.13.00 06/09
07/31/2018 16:49

Test	AC	Time
01 Room Air	0.000	16:50
02 Subject Test 1	0.124	16:51
03 Room Air	0.000	16:51
04 Reported AC	0.124	16:51

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = MIC Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 21
Expiration Date = 08/01/2018
County = 08 Oper No. = 777777


Operator Signature
DEB KASHUR

Remarks:

Form 106-18000

ADDITIONAL TESTS

At some time, certified Intoxilyzer® 8000 operators will need to run additional tests and/or enter important data into the instrument. The operator will go to Level 1 to run the following tests/checks when needed:



C = Calibration Check (ACA)

P = Print Test

S = Configure Standard

Q = Quit

Calibration Check (ACA)

ACA = Air Blank, Calibration Check, Air Blank.

The Ethanol Gas Standard serves as a control to check if the Intoxilyzer® 8000 is calibrated and functioning properly to analyze breath samples accurately at the time of breath testing. The standard is analyzed:

1. Between the first and second breath sample of each subject test in the Custom Mode Sequence.
2. Each time a gas cylinder is replaced due to the gas standard being expired or empty.
3. Each time the Intoxilyzer® 8000 is installed by a Field Inspector.

This section will concentrate on replacing the gas cylinder and running an ACA test.

Ethanol Gas Standard:

The Ethanol Gas Standard simulates the equivalent of 0.080% vapor-alcohol (0.080 g/100 mL of blood or 0.080 g/210 L of end expiratory breath).

Any *certified* chemical test operator can install a new Ethanol Gas Standard and run an ACA test when the cylinder is empty or expired. The Crime Laboratory cannot mail the gas cylinders so officers need to pick up a cylinder at the Lab (please call ahead).

The operator first needs to print a copy of the [Ethanol Gas Standard Cylinder Report SFN59282](#) and the [Intoxilyzer® 8000 Record SFN50496](#) found on the ND Attorney General website @ <http://www.ag.nd.gov/CrimeLab/BreathAlcoholProgram/OperForms/OperForms.htm>.

Follow the instructions on the form.

The gas standard information needs to be scanned/entered into the Intoxilyzer® 8000. Do this by pressing **Escape, Escape, Enter** to get into Level 1, then “**S**” to configure the standard.



Either press “**S**” then Enter or use the arrow key to move to “**S**” and press Enter.



D = Dry Cal Check, W = Wet Cal Check, I = Internal Cal Check

Choose “**D**” then Enter.



Either scan the cylinder barcode or manually enter cylinder information.

For the cylinder on the right:
Lot # = 16512080A2
Expiration Date = 8/1/2014
Cylinder # = 019



The updated cylinder data is being saved. **Attach the gas cylinder to the gas regulator on the Intoxilyzer® 8000 and lock the compartment. Now run the ACA test.**

Begin ACA Test:

If the instrument went back to Ready Mode, begin by pressing **Escape, Escape, Enter**. If the instrument is still in Level 1, proceed as follows:



Either press key **C** then Enter or use the arrow keys to move over to “**C**” on the display and then Enter.

Scan operator ID card, verify information displayed, and press Enter to accept data and move forward.

Enter the two digit county code.

The instrument will perform a Room Air, then a Reference.



The standard gas will be analyzed three consecutive times and each sample should fall in the range of 0.075 - 0.085 AC. **Sign test record.** See test example below.

ACA Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = BSPD 8164.13.00 06/09
07/29/2018 10:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:20
02 Std. Gas	0.079	10:20
03 Room Air	0.000	10:21
04 Std. Gas	0.079	10:21
05 Room Air	0.000	10:22
06 Std. Gas	0.078	10:22
07 Room Air	0.000	10:23

Lot No = 16512080A2
Cyl No = 21
Exp Date = 08/01/2018
County = 08 Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:
ACA test

Form 106-I8000

Print Test

P = Print Test

An operator may need to run a print test when trouble shooting external/internal printer issues. A print test is also the simplest and quickest test to determine if the operator running the test has their operator information uploaded into that Intoxilyzer® 8000 by the Crime Lab. If not, the instrument should display “Invalid ID” and will not proceed further.

Begin Print Test:

Begin by pressing **Escape, Escape, Enter** to get into Level 1, then “**P**” to run a print test.



Either press key P then Enter or use the arrow keys to move over to “P” on the display and then Enter.

Scan operator ID card, verify information displayed, and press Enter to accept data and move forward.

See the following page for example of Print Test.

WHAT IS WRONG WITH MY TEST?

Intoxilyzer® 8000 operators may run into situations that may affect their test record. If this occurs, the Intoxilyzer® 8000 is designed to abort a test if it detects an incident that may affect the test result. What the operator does next is critical to obtaining a valid test.

This section will cover what factors may cause such incidents and what to do if they arise. Try not to get upset as the Intoxilyzer® 8000 is doing what it is designed to do.

The following pages are examples of test records one may see in the field that would make the test invalid.



RFI

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = BSPD 8164.13.00 06/09
08/13/2018 04:26

Test	AC	Time
01 Diagnostic	OK	04:27
02 Room Air	0.000	04:27
03 Subject Test 1	0.089	04:28
04 Room Air	0.000	04:30
05 Std. Gas	0.079	04:32
06 Room Air	0.000	04:33
07 Subject Test 2	0.088	04:33
08 Room Air	RFI*	04:36
09 Room Air	0.000	04:37

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 21
Expiration Date = 08/01/2018
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:

RFI

Form 106-I8000

WHY? A strong level of radio frequency interference (RFI) is present and may give a false breath result or gas standard analysis.

WHAT DO I DO? This test record is invalid, but it still needs to be signed and turned in with any other tests run (breath, blood, or urine). Try to remove the source of RFI and repeat the test as soon as possible. Removing RFI could be as simple as turning off your radio or cell phone, or it may be more difficult, such as in a correctional facility where RFI's are more common due to increased radio activity.

Ambient Fail

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = BSPD 8164.13.00 06/09
08/13/2018 04:16

Test	AC	Time
01 Diagnostic	OK	04:17
02 Room Air	0.000	04:18
03 Subject Test 1	0.032	04:18
04 Room Air	0.000	04:20
05 Std. Gas	0.078	04:21
06 Room Air	0.000	04:23
07 Subject Test 2	0.032	04:23
08 Room Air	AMB*	04:24
09 Room Air	0.000	04:25

*Invalid Test
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 21
Expiration Date = 08/01/2018
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:

ambient fail

Form 106-I8000

WHY? The room air is contaminated with chemicals that may interfere with testing. These chemicals may be smoke, alcohol, cleaning supplies, or even paint fumes.

WHAT DO I DO? Clear the room of any possible contaminants. You may need to open a window or bring a fan into the room. Repeat the test as soon as possible and sign all test records, valid and invalid. If the room does not clear, get blood or urine.

Difference Too Great

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = BSPD 8164.13.00 06/09
08/13/2018 04:05

Test	AC	Time
01 Diagnostic	OK	04:06
02 Room Air	0.000	04:06
03 Subject Test 1	0.031*	04:07
04 Room Air	0.000	04:09
05 Std. Gas	0.079	04:10
06 Room Air	0.000	04:12
07 Subject Test 2	0.087*	04:12
08 Room Air	0.000	04:13

*Difference Too Great


Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 21
Expiration Date = 08/01/2018
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:

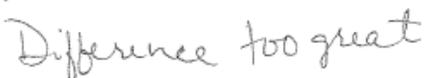

Form 106-I8000

WHY? This results when the difference between the two adequate breath samples is greater than 0.020 AC. It's possible the officer and/or operator didn't wait the required 20 minute deprivation period. Or maybe the subject regurgitated alcohol back up into the esophagus or mouth. This may give a false high result.

WHAT DO I DO? Wait another 20 minutes with the subject having nothing to eat, drink, or smoke. Repeat the test. Sign all records, valid and invalid. Another option is to get blood or urine (attach your breath alcohol test record).

Invalid Test – Mouth Alcohol

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = BSPD 8164.13.00 06/09
08/13/2018 05:30

Test	AC	Time
01 Diagnostic	OK	05:31
02 Room Air	0.000	05:32
03 Subject Test 1	0.135	05:32
04 Room Air	0.000	05:34
05 Std. Gas	0.077	05:36
06 Room Air	0.000	05:37
07 Invalid Sample	X.XXX	05:38
08 Room Air	PUR*	05:38

*Purge Fail
**Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 21
Expiration Date = 08/01/2018
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:

Mouth alcohol (and purge fail)
Form 106-I8000

WHY? The instrument detected residual mouth alcohol on the subject’s breath sample.

WHAT DO I DO? Wait another 20 minutes with the subject having nothing to eat, drink, or smoke. Repeat the test. Sign all records, valid and invalid. Another option is to get blood or urine (attach your breath alcohol test record).

Note: There is also a Purge Fail on this test record. This means that the instrument did not adequately purge the sample chamber of ethanol during the timed room air. This is not a valid test. Sign the prior test record and try running another test. If there are still problems after repeating the test, contact your Field Inspector or the Crime Laboratory to troubleshoot.

Calibration Out of Tolerance

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = BSPD 8164.13.00 06/09
08/13/2018 05:11

Test	AC	Time
01 Diagnostic	OK	05:12
02 Room Air	0.000	05:12
03 Subject Test 1	0.127	05:13
04 Room Air	0.000	05:15
05 Std. Gas	0.073*	05:17
06 Room Air	0.000	05:17

*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 21
Expiration Date = 08/01/2018
County = 08 Oper No. = 77777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:

Calibration check out of tolerance

Form 106-I8000

WHY? The calibration check is out of the tolerance range of 0.075-0.085 AC.

WHAT DO I DO? Get blood or urine. Don't waste time trying to run another breath test, but do sign this test record and turn it in with your alternative test results. The instrument may need to be recalibrated. When possible, you or the Field Inspector should call the Breath Alcohol Division at the Crime Lab to troubleshoot.

Interferent Detected

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = BSPD 8164.13.00 06/09
08/13/2018 03:54

Test	AC	Time
01 Diagnostic	OK	03:55
02 Room Air	0.000	03:55
03 *Subject Test	INT*	03:56
04 Room Air	0.000	03:56

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 21
Expiration Date = 08/01/2018
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:



Form 106-18000

WHY? The subject's breath sample contains a substance other than ethanol, such as, acetone, methanol, or other denatured alcohols. The subject may be having a diabetic reaction or has been drinking alcohol deemed not fit for human consumption.

WHAT DO I DO? Get the subject medical attention! You will not be able to continue with a breath test, so get blood or urine at the medical facility when able.

Refusal

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = BSPD 8164.13.00 06/09
08/13/2018 03:50

Test	AC	Time
01 Diagnostic	OK	03:50
02 Room Air	0.000	03:51
03 *Subject Test	REF*	03:52
04 Room Air	0.000	03:52

*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 21
Expiration Date = 08/01/2018
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Refusal from subject

Form 106-18000


Nothing is “wrong” with this test. However, this invalid test may be used in legal proceedings to show a refusal by the subject to provide an adequate breath sample. Here, an operator pressed the “R” key on the keyboard because the subject failed to cooperate during the testing procedure. Don’t forget to sign it.



Remember, if you already have one adequate breath sample from the subject you may want to continue with the test so you have a Reported AC. You will not get a Reported AC if you press the R key for a refusal.

TROUBLE SHOOTING & QUESTIONS

Operator Cards:

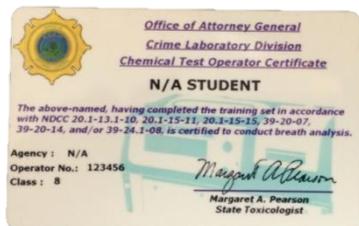

Q. I lost my operator card. Can I still run a test on the Intoxilyzer® 8000?

A. Yes. You may manually enter your operator information. Contact the Crime Lab for another card.

Q. When I scan my operator card or manually enter my operator information the displays reads INVALID ID and

won't let me continue to run a test.

A. If you are manually entering your information, first make sure you entered your operator number correctly. If you have entered it correctly and/or your operator card fails to work you may have let your certification lapse and the instrument no longer recognizes your information. Take online recertification training at:

<http://www.ag.nd.gov/CrimeLab/LawEnforcementTraining/BreathAlcohol/index.htm>.

Also, the Intoxilyzer® 8000 may not have the most current download of operator information. Contact the Breath Alcohol Program to update data, if needed.

Note: Once the display reads INVALID ID it will go back to display, Please scan ID or press enter. To go back to Ready Mode press the Start Test button. Sequence Aborted will display and will then go back to reading Ready Mode for another operator to run a test.

Testing:

Q. The Intoxilyzer® 8000 keeps running a single breath test when I want to run a two breath test. What's wrong with the instrument?

A. Nothing is wrong with the instrument. You're in ABA mode (Esc, Esc, Enter) when you need to get into CMS (Esc, Esc, Start Test). Even though the ABA test may be a good test for a different situation, the Approved Method to Conduct Breath Tests with the Intoxilyzer® 8000 was not followed, therefore, making the ABA test an invalid test in court. *Remember to sign and submit all test records (valid and invalid).*



Q. Instead of waiting the 20 minute deprivation period with the subject having nothing to eat, drink, or smoke I let the subject blow closer to 15 minutes because I was in a hurry. Is it really that big of a deal?

A. Yes, it is a big deal! This doesn't follow the Approved Method to Conduct Breath Tests with the Intoxilyzer® 8000. And, a 20 minute wait further ensures any possible mouth alcohol has dissipated. Some states may follow a 15 minute deprivation period, but we are not one of them.

Printer:



Q. The Intoxilyzer® 8000 is printing from the internal printer instead of the external printer. How do I fix this?

A. First of all, I want to stress that a test record printed by either printer can be legally used. But, to print again with the external printer, in most cases the Intoxilyzer® 8000 will need to be rebooted so it can recognize the external printer again. Do this by turning off both the main power and battery switches on the back of the instrument.

Make sure the external printer's power switch is on, all printer connections are properly seated, and printer paper is inside. Power the Intoxilyzer® 8000 back on by first turning on the main power switch, listen carefully for a "click" and then turn on the battery switch. After a brief warm-up the Intoxilyzer® 8000 will be back in Ready Mode. To check if it will print externally, run a Print test (Esc, Esc, Enter, P).

Q. How do I print another copy of my test record?

A. You can print another copy of only the most recent CMS test record by pressing the F1 key.

Q. I lost the test record(s) and I have court coming up. Can I get a copy of my test from the Crime Lab?

A. No. We download Intoxilyzer® 8000 data for statistical purposes only. The data we see at the lab looks nothing like the test record (Form 106 - I8000) and cannot be used in court.

